

Directions for Rectal Use

NexTemp® Plus


MEDICAL
INDICATORS




Open the package and remove one thermometer from the dispenser on the top of the box. Do not remove the plastic sheath.



Lubricate the thermometer and insert into the rectum until the dot matrix and numbers are completely covered.

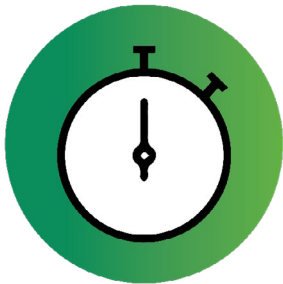

Wait 3 minutes before withdrawing the thermometer and sheath from the rectum (discard the sheath immediately).

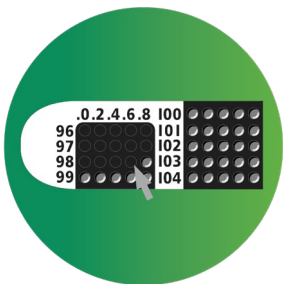

To read the temperature: the last black dot on the matrix indicates the correct temperature for NexTemp® Plus thermometers.



The temperature will remain visible for 20 seconds with NexTemp® Plus thermometers. Once you have read and recorded the temperature, discard the thermometer.